


MS16 User Manual

The MS16 Baby Scale from Brecknell has been designed to quickly and easily gather weight and height readings on the same scale. Equipped with an easy-to-use weight hold function, the MS16 is ideal for babies that are hard to keep still.

Before using your MS16 Scale, please install the 4 AA batteries supplied with the scale. These batteries need to be installed into the battery compartment found on the underside of the scale.

The MS16 Scale comes with 4 self-leveling feet. For best accuracy, place the scale on a safe, hard, flat, level surface before turning on.

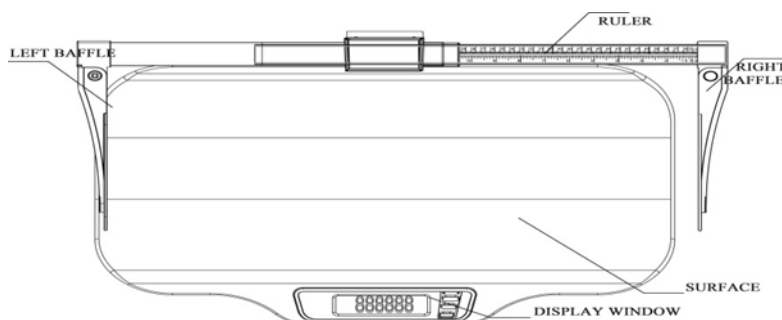

This scale has a maximum capacity of 44lb x 0.01lb (20kg x 5g) with a height range from 14 inches to 27.5 inches (35cm to 70cm).

1.1 Scale Functions

ON/TARE

Turns on scale / Tares off any weight shown on the weight display.
Long press for 3 seconds will turn off the scale.

MODE

Press **MODE** to switch between kg and lb.

MEM

Used to store a weight reading to memory so it can be easily recalled when there is no weight on the scale.

To use this memory function, the weight must be over 1.75 lb (800g) with the scale turned on.



1.2 Scale operation

1. Turn on the scale, making sure the displayed reading is showing zero (0.000).
2. If using a protective sheet or blanket on the scale pan, place this on the scale first and press the **ON/TARE** key to zero off this weight.
The display should now show (0.000).
3. Gently place the baby on the scale pan and wait for the display reading to stabilize.
Once stable, the required stable weight reading will flash.
4. Record the weight reading or press the **MEM** key before removing the baby from scale.
The scale returns to zero.
5. If the **MEM** key was pressed, press it again to recall the last stable weight reading.
Scale will automatically turn off after 5 minutes if not used.

Calibration instructions for this scale, if needed, can be found at:

<http://www.brecknellscales.com/doc.html>

Brecknell USA
1000 Armstrong Dr.
Fairmont MN 56031
Toll free:800-637-0529
Tel:507-238-8702
Fax:507-238-8271
Email:sales@brecknellscales.com
<http://brecknellscales.com>

Brecknell UK
Foundry lane,
Smethwick, West Midlands
England, B66 2LP
Tel:+ 44 (0) 8452 466717
Fax:+44 (0) 8452 466718
Email:sales@brecknellscales.co.uk
<http://brecknellscales.com>

